



Email : sales@naugramedical.com
Phone: 9896600003

Product Name :
Plain Hot Water Bottle

Product Code :
NMCBS35002



Description :

Plain Hot Water Bottle We are among the reputed organizations, engaged in offering our clients with Plain Hot Water Bottle. All the beds are designed in compliance with international quality norms & standards, using premium raw material, which is sourced from the reliable and certified vendors of the market. Apart from this, we have a team of expert quality controllers, which thoroughly checks all the products on various parameters to ensure flawlessness. Instruction for use: · Unscrew the stopper · Fill warm water up to two third of the capacity · Expel the air by pressing the bottle gently · Put the stopper and tight it · Bottle is ready for use Precautions: · Do not use boiling water · Keep one third of the bottle empty · After filling press gently to expel air · Stopper should not be loose · When not in use keep the empty bottle upside down after removing the stopper · Avoid mishandling by children Plain Hot Water Bottle, Hospital Rubbers Goods exporters, Hospital Rubbers Goods manufacturers



Medical Hospital Equipments Manufacturers

Website: [SITE](#)