

Email: sales@naugramedical.com

Phone: 9896600003

Product Name:

Hot Water Bottle Both Side Ribbed

Product Code: NMCBS35004



Description:

Hot Water Bottle Both Side Ribbed \cdot Unscrew the stopper \cdot Fill warm water up to two third of the capacity \cdot Expel the air by pressing the bottle gently \cdot Put the stopper and tight i \cdot Bottle is ready for use Precautions: \cdot Do not use boiling water \cdot Keep one third of the bottle empty \cdot After filling press gently to expel air \cdot Stopper should not be loose \cdot When not in use keep the empty bottle upside down after removing the stopper \cdot Avoid mishandling by children Hot Water Bottle Both Side Ribbed, Hospital Rubbers Goods exporters, Hospital Rubbers Goods manufacturers



Medical Hospital Equipments Manufacturers

Website: STE

1/1