



Email : sales@naugramedical.com  
Phone: 9896600003

**Product Name :**  
Hot Water Bottle One Side Ribbed

**Product Code :**  
NMCBS35003



**Description :**

Hot Water Bottle One Side Ribbed Being a customer focused organization; we are engaged in offering a wide and exotic assortment of Hot Water Bottle One Side Ribbed. These are designed as per the specifications laid down by the customers for meeting their needs. All our products are available in different models and designs at market leading prices to fulfill the requirements of customers. Instruction for use: · Unscrew the stopper · Fill warm water up to two third of the capacity · Expel the air by pressing the bottle gently · Put the stopper and tight it · Bottle is ready for use Precautions: · Do not use boiling water · Keep one third of the bottle empty · After filling press gently to expel air · Stopper should not be loose · When not in use keep the empty bottle upside down after removing the stopper · Avoid mishandling by children Hot Water Bottle One Side Ribbed, Hospital Rubbers Goods exporters, Hospital Rubbers Goods manufacturers



**Medical Hospital Equipments Manufacturers**

**Website:**Medical Hospital Equipments Manufacturers,